Wrist Electronic Blood Pressure Meter

USER MANUAL

Dear wrist electronic blood pressure monitor users:

Thank you for purchasing our wrist electronic blood pressure monitor. In order to operate this machine accurately, please read this operating instruction carefully before using.

Please put the instructions on the side in order to check.


Please take good care of, do not lose it.

Content

1. Product Structures and Settings	3
2. Measurement Procedures	5
3. Symbols and Error Messages	7
4. Maintenance	9
5. Product Specifications	10
6. Attention	11

PAGE 2

1. Product structures and Settings

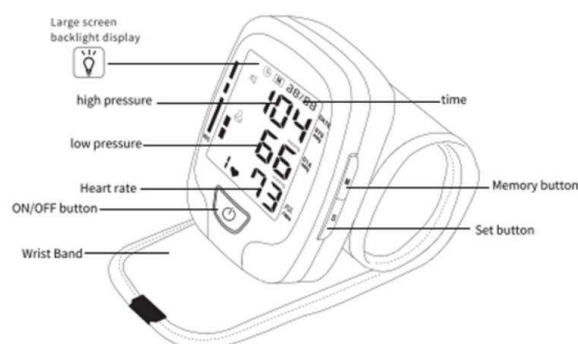Install the Battery

- 1、Open the battery cover by pushing it in the direction indicated on the cover.
- 2、Install 2 “AAA” batteries so the polarities match the polarities of the battery compartment indicated on the device.

Note:

- 1、If the unit will not be used for more than 6 months, remove all the batteries.
- 2、If shown on the display, please install 2 new batteries at the same time.
- 3、Memory of previous reading will remain after changing the batteries.

Warning:

- 1、It is better to use batteries of the same brand or the same type.
- 2、Do not throw your batteries into the fire, they may explode.
- 3、Please handle.
- 4、If battery fluid should get in your eyes, immediately rinse with plenty of clean water. Contact a physician immediately.
- 5、Disposal of batteries according to the appropriate local regulation. Unlawful disposal may.

PAGE 3

Setting your device

1. Select the user

Press SET button for at least 6 seconds, symbol flashes. Press MEM button to select between user 1 and user 2. Press SET button to confirm.

2. Set the Date

After select the user, Year displays and flashes. Press MEM button to increase the number by increments of one (hold MEM button to increase faster). Press SET button to set the year, then Month display and flashes. Follow the procedure above to set the Month, Day, Hour, and Minute in order.

3. Switch On/Off the Voice

After setting the Minute, SP ON or SP OFF displays and flashes. Press MEM button to switch. SP ON indicates that the voice function is on, and SP OFF is voice off. Press SET button to confirm.

4. Switch the Pressure Unit

After setting the Voice, PR ON or PR OFF displays and flashes. Press MEM button to switch. PR ON is kPa, PR OFF is mmHg. Press SET button to confirm.

PAGE 4

2. Measurement Procedures

Apply the Wrist Cuff

1. Remove any cloth that covers your wrist. If you roll up your sleeve, do not roll up too tightly, that may constrict the blood flow in your arm.
2. Wrap the wrist band around your left wrist firmly, leaving 1-2cm between the cuff and the bottom of your palm. The monitor screen should face the same direction as your palm does. (If your heart is on the right side, wrap the wrist band around your right wrist).


Taking a Measurement

Prepare your measurement


1. Position yourself on a chair comfortably with your back straight and feet on the ground.
2. Rest for at least 5mins, do not smoke, drink stimulating beverages, such as coffee and alcohol. Relax for at least 30mins after intensive exercise.
3. Put your arm near your heart as the picture shown below, relax your whole arm and have your other arm to support. Do not move your body during measurement, especially the arm being measured.

PAGE 5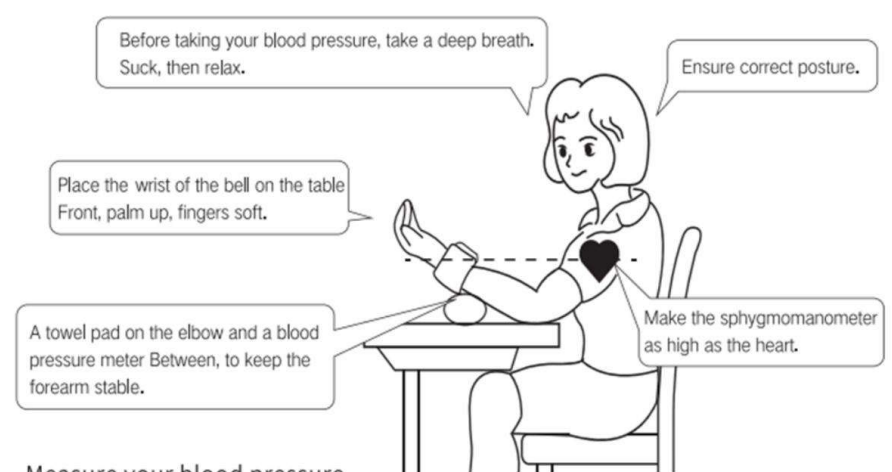

Measure your blood pressure

When you are ready, press the ON/OFF button. The device will start to work automatically. Wait until the measurement completes.

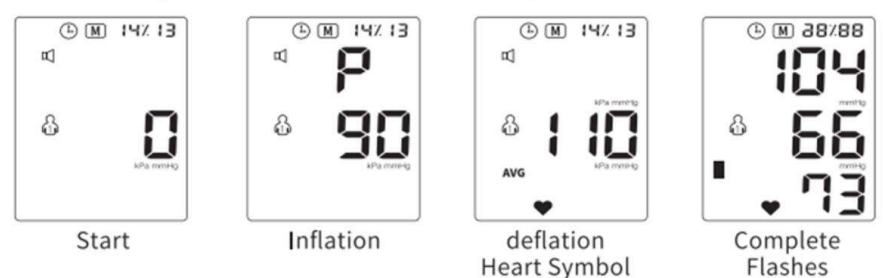PAGE 6

Memory Function

The monitor can automatically store up to 99 set of last measurement reading each for 2 users. When memory reach 99 readings, the new reading will cover the oldest reading.

Review your result

1. Select the User. please refer to section 1.
2. Press MEM button, your last measurement result displays on the screen. The first reading shown will be the average of your last 3 results. Press MEM button again, the monitor displays previous result from the recent to the older.
3. When measure more than 99 times, the No 99 reading will be deleted and the new reading will be stored.

Delete Memory

1. Delete a single reading
When the reading you want to delete shown displays, hold the SET button, "DEL" should display on the upper right corner, press MEM button to delete.
2. Delete entire readings
Hold the MEM button for at least 6 seconds (do not let loose), press SET button, then displays, stop holding MEM and SET buttons together, all readings will be deleted.

Discontinue a Measurement

If it is necessary to interrupt a blood pressure measurement for any reason (e.g. the patient feels unwell), the Start/Stop button can be pressed at any time. The device then immediately lowers the cuff pressure automatically.

Note

1. Rest for at least 5 mins between every measurement.
2. Put your monitor into the protection case after measurement.
3. Store and operate your monitor under our recommended environment. Please refer to section 5 Product Specifications.

PAGE 7

3. Symbols and Error Messages

Irregular Heartbeat Indication

When the monitor detects irregular rhythms, the Irregular Heartbeat Symbol (⚡) will display on the lower left corner of the screen.

An irregular heartbeat rhythm is a rhythm that varies by less than 25% from the average rhythm or 25% more than the average rhythm detected while the monitor is measuring the systolic and diastolic blood pressure. If the irregular heartbeat symbol displays, please consult your physician.

WHO Blood Pressure Indication

The monitor has WHO blood pressure indicator displayed on the screen. Please refer to the picture below for understanding the meaning of indications. This is served as a reference of how healthy your blood pressure is. Please consult your physician for any changes of your blood pressure.

Range	Systolic pressure (SYS)	Diastolic pressure (DIA)	Suggest corresponding measures
—	<100	<60	Please check with a doctor
==	100-120	60-80	Self testing
===	120-139	80-89	Self testing
====	150-159	90-99	Consult a doctor
=====	160-179	100-109	Consult a doctor
=====	≥180	≥110	Please see a doctor as soon as possible

PAGE 8

Error Messages

The following symbol will appear when the measurement is abnormal

LCD error prompt	
ER1:	1. Determine whether the gas in the wristband has been exhausted before measurement 2. Do not squeeze the wrist strap after the measurement
ER2:	Do not move your wrist during the measurement
ER3:	Abnormal measurement results
ER4:	The wristband is too loose or leaking (the pressure is less than 30 mmHg within 10 seconds)
ER5:	The trachea is blocked
ER6:	Excessive interference during measurement
ER7:	Pressure exceeds upper limit

Trouble removal

SYMBOL	Check	Cause and solutions
No power	Check the battery power	Replace new one
	Check the polarity position	Installation for proper placement of the batteries polarities
No inflation	If the air plug inserted securely	Insert into the air socket tightly
	If the plug broken or leak	Change a new cuff

PAGE 9

Err and stop working	If the arm moved when inflate	The arm should remain still
	If chatting during measurement	Do not talk when measure
Cuff leak	If the cuff wrapped too loosely	Wrap the cuff tightly
	If the cuff broken	Change a new cuff
⚠ Please contact the distributor if you can't solve the problem, do not disassemble the unit by.		

1. Wash hands after each time measurement. If one device is used by different patients, wash hands before and after each use.
2. Do not expose the device to either extreme temperatures, humidity, dust or direct sunlight.
3. The cuff contains a sensitive air-tight bubble. Handle this cuff carefully and avoid all types of stress through twisting or buckling.
4. Clean the device with a soft, dry cloth. Do not use gas, thinners or similar solvents. Spots on the cuff can be removed carefully with a damp cloth and soap suds.
5. Handle the tube carefully. Do not pull on it. Do not allow the tubing to kink and keep it away from sharp edges.
6. Do not drop the monitor or treat it roughly in any way. Avoid strong vibrations.
7. Never open the monitor. This invalidates the manufacturer's warranty.
8. Batteries and electronic instruments must be disposed of in accordance with the locally applicable regulations, not with domestic waste.
9. Remove the battery if the monitor will not be used for three months or longer.

Accuracy test

Sensitive measuring devices must be checked for accuracy from time to time. We recommend a periodical inspection of your monitor by an authorized dealer every year. Please turn to local distributor or the manufacturer.

- Caution:
1. Change or modification without approval will void the user warranty. Do not disassemble or repair any component.
 2. Do not submerge the monitor and the cuff in water.

PAGE 10

☔ Protect the device from water and moisture
☀ Protect the device from direct sunlight
❄ Protect the device from extreme heat and cold
📶 Avoid proximity to electromagnetic fields, such as those produced by mobile phones
🔧 Never open device
🛡 Protect device from impact and drops
Protect the device from extreme humidity and altitude

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